

SOMNI
VETERINARY



SOMNI REBREATHING ANESTHESIA SYSTEM ASSEMBLY GUIDE

► EQUIPMENT NEEDED:

4 mm Allen wrench


Pre-assembled base and screws inserted into the holes they are designed for.



1

STEP 1:

Remove Base from the shipping box. The concentrator plate and wheels will be pre-assembled with the back support bracket to the base plate frame.

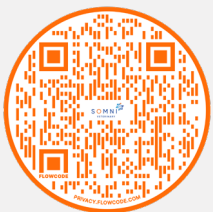

1-877-637-3625

info@somnivets.com

www.somnivets.com

Proudly machined, assembled, and calibrated in the USA

2

STEP 2:

Stabilize the middle shelf, insert into riser post. Fasten the three screws in the riser post.



3

STEP 3:

Insert riser post into back of support bracket, fasten 2 screws on each side.



4

STEP 4:

Mount face plate to the riser post. Fasten two screws on each side.



1-877-637-3625

info@somnivets.com

www.somnivets.com

Proudly machined, assembled, and calibrated in the USA