


INSTALLATION INSTRUCTIONS EPS-3 EXPOSURE PREVENTION SYSTEM

Place the SOMNI EPS-3 on a flat, stable surface and ensure the Main Power/Timer Switch, and flowmeter controls are in the “OFF” position.

Setup for use with an extended capacity passive scavenging filter or non-recirculating fume hood:

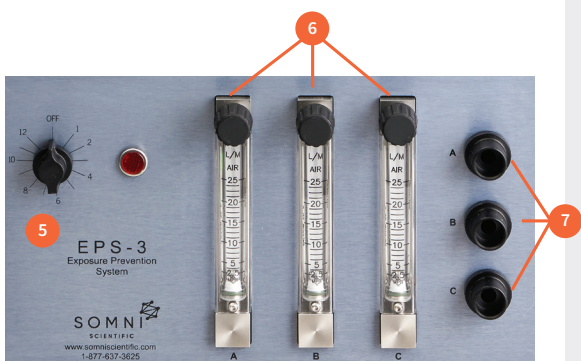

1. Connect the 19mm or ¼” ID tubing with 15mm end to the W.A.G. canister inlet located at the top of the canister
2. Connect the other end to the EPS-3 waste gas outlet located on the back. Keep filter upright at all times!
3. If discharging the waste gas from the SOMNI EPS-3 into a non-recirculating vent, downdraft or other installed, active method of waste gas removal, place tubing connected to waste gas evacuation port at least 6 inches into waste gas vent. *(Not Pictured)*
4. Plug SOMNI EPS-3 into appropriately grounded electrical outlet.
5. Turn Main Power/Timer Switch to desired time setting, this will turn ON the negative flow generating pump inside the SOMNI EPS-3 and automatically turn OFF after the selected time frame has passed.
6. Set each of the flowmeters to the appropriate vacuum levels.
7. Connect each of the anesthetic supply accessories’ scavenging line to the ports on the right-hand side, labelled “A”, “B” or “C”.
8. Open the stopcock at each of the scavenging lines desired. Verify vacuum is occurring at each of the scavenging lines. *(Not Pictured)*

Setup for use with an in-house vacuum system, EPS-3 Non Pump Version (WG-15002):

- Connect Vacuum line from in-house vacuum system to the waste gas evacuation port in the back of the EPS-3. Verify In-house vacuum system is in “On” position. *(See #2 Photo)*
- Set each of the flowmeters to the appropriate vacuum levels. *(See #6)*
- Connect hose/tubing from anesthetic supply accessories (induction chamber, nosecone) to the evacuation ports labeled “A”, “B”, and/or “C”. Open stopcock on each scavenging line intended for use. *(See #7)*
- Verify the stopcock for each scavenging line is open when in use. *(Not Pictured)*

*****DO NOT OPERATE EPS-3 WITH FLOWMETERS IN THE OFF POSITION.**

Suggested Negative Flowrates for Accessories:

Uniflow 7-10 lpm
Miniflow 7-10 lpm
Chamber < 15 lpm

**COMPLETE OPERATIONS MANUAL CAN BE
DOWNLOADED AT SOMNISCIENTIFIC.COM**

SOMNI PROVIDES UNPARALLELED CUSTOMER SERVICE, CLINICAL AND TECHNICAL SUPPORT.

For more information or pricing, please contact our sales team.

☎ 1-877-637-3625 ✉ info@somniscientific.com

🌐 www.somniscientific.com



*Proudly machined, assembled
and calibrated in the USA*