

OPERATING INSTRUCTIONS W.A.G. WASTE ANESTHETIC GAS FILTER



1. Weigh the W.A.G. filter prior to use and record starting weight as shown.
2. Place the filter on a flat surface upright so the vents in the top are clear of any obstruction. The W.A.G. filter must be placed at or below the level of anesthetic source for passive waste gas collection.
3. Attach waste gas tubing from anesthetic source. The W.A.G. filter accommodates 15mm adapter, 19mm tubing or 22mm tubing.
4. Anesthetize the animal following standard protocol.
5. When use of anesthetic is complete, continue flowing oxygen/ carrier gas for several minutes to allow all remaining anesthetic gas to be removed from the system and captured in the W.A.G. Filter.
6. When W.A.G. filter gains 200 grams of weight, discard following standard protocol and replace.



SOMNI PROVIDES UNPARALLELED CUSTOMER SERVICE, CLINICAL AND TECHNICAL SUPPORT.

For more information or pricing, please contact our sales team.

☎ 1-877-637-3625 ✉ info@somniscientific.com

🌐 www.somniscientific.com



*Proudly machined, assembled
and calibrated in the USA*